## **Wrist Watch Blood Pressure Monitor Model M2-W**

### **INSTRUCTION BOOK**

Thank you for choosing our Wrist Watch Blood Pressure Monitor Gamma™ designed on the basis of the most advanced technologies. The device you are holding now is your reliable assistant. Thanks to its portability and user-friendliness you can measure your blood pressure with this monitor whenever and wherever you like: during your travel, at work, on holidays, in the gym.

We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Gamma Trade Mark.

Before starting to use this device please study the instruction book carefully. The instruction book offers all information you need to correctly measure your blood pressure and pulse.

If you suffer from an irregular heartbeat, measurements taken with this instrument should only be evaluated after consultation with your doctor.

The wrist watch blood pressure monitor is best for:

- people in their young and middle age if they do not suffer from arterial sclerosis;
- people who always need to have a blood pressure monitor with them and who need to often control their blood pressure.
- people who face difficulties with fitting the cuff on the upper arm correctly because of their overweight.

Regular blood pressure control allows for reliable and easy health monitoring and for effective combating possible complications caused by blood pressure abnormalities.

#### **Attention**

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

## CONTENT

### 1. Features of Wrist Watch Blood Pressure Monitor M2-W

- 1.1. Components of your blood pressure monitor kit
- 1.2. Description of the blood pressure monitor

### 2. Getting ready for measurement

- 2.1. Inserting the batteries
- 2.2. Setting the time and date

### 3. Measurement Procedure

- 3.1. Fitting the cuff
- 3.2. Measuring procedure
- 3.3. Reading measurement results
- 3.4. Discontinuing a measurement
- 3.5. Irregular Heartbeat Detector

### 4. Memory function

### 5. Error Messages/Troubleshooting

### 6. Care and maintenance

### 7. Technical specifications

### 8. Warrantee

### 9. Important information on blood pressure and its measurement

- 9.1. What is blood pressure?
- 9.2. Which values are normal?
- 9.3. What can be done to lower/maintain the level of blood pressure?

## 1. Features of Wrist Watch Blood Pressure Monitor M2-W

### 1.1. Components of your blood pressure monitor kit

The blood pressure monitor kit includes Wrist Watch Blood Pressure Monitor Model M2-W, 2 batteries (AAA), Storage Case, Instruction Book, Warranty Card, kit box.

### 1.2. Description of the blood pressure monitor

## 2. Getting ready for measurement

### 2.1. Inserting the batteries

To insert batteries

1. Remove cover as illustrated.
2. Insert the batteries (AAA, 1.5 V), following the indicated polarity. (+ -)

### ATTENTION

- If the battery warning  appears in the display, the batteries are almost flat and must be replaced by new ones
- After the battery warning  appears, the device is blocked until the batteries have been replaced.
- If the blood pressure monitor is not used for long periods, remove the batteries from the device.
- Please use "AAA" Long-Life or Alkaline 1.5V Batteries. Do not use rechargeable batteries.
- Functional check: Hold the START button down to test all the display elements. When functioning correctly many icons will appear.


### 2.2. Setting the time and date

This blood pressure monitor incorporates the function of time and date display. Time and date are recorded along with pressure and pulse.

The clock runs from the following setting: year-month-day 00:00 hour.

1. After the new batteries are fitted, the year number flashes in the display.

2. You can set the year by pressing the M-button. To confirm and then set the month, press the time button.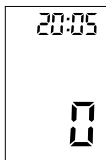


3. You can now set the month using the M-button. Press the time button to confirm and then set the day.

4. Please follow the instructions above to set the day, hour and minutes.

5. Once you have set the minutes and pressed the time button, the date and time are set and the time is displayed.

6. If you want to change the date and time, press and hold the time button down for approx. 3 seconds till Year flashes, now you can enter the new values as described above.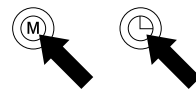

### Note

- 1 x press on the MEMORY button and the TIME button advances one operation (e.g. switch to the next setting mode or change the figure into +1)
- Holding the button down speeds up the procedure.

### 3. Measurement Procedure

#### ATTENTION

- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Always measure on the left wrist.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.



#### 3.1. Fitting the cuff

1. Remove all jewellery and watches. Apply the cuff so that the display is facing you.

2. The distance between the cuff and the hand should be nearly 10 mm.

3. Secure the cuff with the Velcro fastener, so that it lies comfortably and not too tight. A loose cuff causes false measurement values.

4. Lay your arm on a table with your palm upwards. Support your arm with a cushion or the storage case so that the cuff rests at about the same height as the heart. Remain still for 2 minutes, sitting quietly, before beginning a measurement.

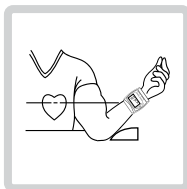
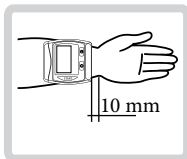
#### 3.2. Measuring procedure

To carry out measurement

1. Press the START button. The built-in microprocessor begins to inflate the cuff. In the display, the increasing cuff pressure is continually displayed.

2. After automatically reaching an individual pressure, the pump stops and the pressure slowly falls. The cuff pressure is displayed during the measurement. When the device has detected your pulse, the heart symbol in the display begins to blink and a beep tone is audible for every pulse beat.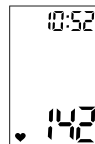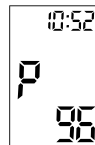


#### 3.3. Reading measurement results

When the measurement has been concluded, a long beep tone sounds. The measured systolic and diastolic blood pressure values, as well as the pulse are now displayed.

The appearance of this symbol  signifies that an irregular heartbeat was detected. This indicator is only a caution. It is important that you be relaxed, remain still and do not talk during measurements. NOTE: We recommend contacting your physician if you see this indicator frequently.

#### Note

To prolong the batteries' life the device switches off automatically if no button is pressed for 1 minute. Otherwise you can switch it off by pressing the START button.

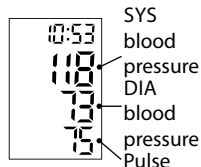SYS  
blood  
pressure  
DIA  
blood  
pressure  
Pulse

### 3.4. Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason the START button can be pressed at any time. The device then immediately lowers the cuff pressure automatically.

### 3.5. Irregular Heartbeat Detector

This function indicates allorhythmic heartbeating. If the symbol of IHD  appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement. In this case, the result may deviate from your normal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol  appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

#### Information for the doctor on frequent appearance of the Irregular Heartbeat Detector


This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The instrument is clinically tested.

The symbol  is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol  more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice.

The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

### 4. Memory function

Wrist Watch Blood Pressure Monitor Gamma™ Model M2-W automatically stores 99 measured results together with the date and the time when they are obtained.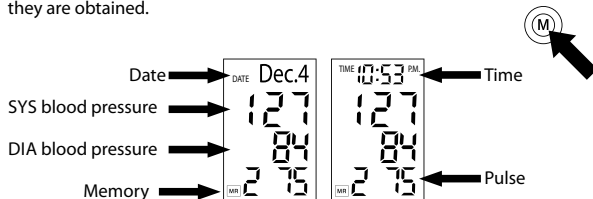

To see the results of the last measurement press and hold the MEMORY button for at least 3 seconds. Each time you press the memory button an earlier measurement will be displayed along with time and date (with symbols M2-M99).


If an irregular heartbeat interval was detected in any measurements stored in memory, the icon  is displayed.

#### Clear all values

If you are sure that you want to permanently remove all stored values, hold down the M-button (the instrument must have been switched off beforehand) until CL appears and then release the button. To permanently clear the memory, press the M-button while CL is flashing. Individual values cannot be cleared.

### 5. Error Messages/Troubleshooting

If an error occurs during a measurement, the measurement is discontinued and a corresponding error code is displayed (picture: error no. 2)

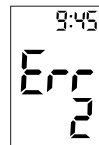Error No.	Possible cause(s) / Solutions
ERR 1	No pulse has been detected. Cuff may be positioned incorrectly.
ERR 2	The arm was moved during the measurement. Repeat the measurement, keeping your arm still.
ERR 3	Inflation of the cuff took too long. The cuff is not positioned correctly. Check that the cuff is correctly connected and is not too loose.
ERR 5	The measuring signals are inaccurate and no result can therefore be displayed. Read through the checklist for performing reliable measurements and then repeat the measurement.*
HI	The pressure in the cuff is too high (over 300 mmHg) OR the pulse is too high (over 200 beats per minute). Relax for 5 minutes and repeat the measurement.*
LO	The pulse is too low (less than 40 beats per minute). Repeat the measurement.*

\*Please consult your doctor, if this or any other problem occurs repeatedly.

#### Other possible errors and their solutions

If problems occur when using the device, the following points should be checked:

Malfunction	Remedy
The display remains blank when the device is switched on although the batteries are in place.	1. Check battery installation/polarity. 2. Remove the batteries and if the display is unusual, then exchange them for new ones.
Every measurement results in different values, although the device functions normally and normal values are displayed.	Blood pressure changes constantly. The observed readings may accurately reflect your pressure - minor differences in the readings are not abnormal.
Blood pressure values differ from those measured at the doctor's.	Record daily values and consult your doctor. Pressure readings in your doctor's office may be higher due to anxiety.

#### Note

Blood pressure is subject to fluctuations even in healthy people. Please remember that **comparable blood pressure measurements always require the same time and the same conditions! These are normally quiet conditions.** If you follow the correct procedures described above and still get the fluctuations of blood pressure of more than 15 mmHg and/or you repeatedly hear irregular pulse tones please consult your doctor.

#### ATTENTION

Should any technical malfunctions arise in the blood pressure monitor, please contact the point of sale where you bought the device or Service Centre Gamma-Medica. **Never attempt to repair the instrument yourself!**

Any unauthorized opening of the instrument invalidates all warranty claims.

## 6. Care and maintenance

Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.	
Handle the cuff carefully and avoid all types of stress through twisting or buckling in order not to damage the sensitive air-tight bubble.	
Clean the device with a soft, dry cloth. Do not use gas, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. Do not submerge the cuff in water!	
Do not drop the monitor or treat it roughly in any way.	
Never open the monitor! Otherwise the manufacturer's calibration will be spoiled.	

### Note:

According to international standards, your monitor should be checked for calibration every 2 years.

## 7. Technical specifications

Weight	137 g (with batteries and cuff)
Size	83(W) x 76(L) x 66(H) mm
Storage temperature	-5 to +50°C (23°F - 122°F)
Humidity	15 to 85% relative humidity maximum
Operation temperature	10 to 40°C ( 50°F - 104°F)
Display	LCD-Display (Liquid Crystal Display)
Measuring method	oscillometric
Pressure sensor	capacitive
Measuring range:	
- SYS/DIA pressure	30 to 280 mmHg
- Pulse	40 to 200 per minute
Cuff pressure display range	0-299 mmHg.
Memory	Automatically stores the last 99 measurements.
Measuring resolution	1 mmHg
Accuracy	
- pressure	within $\pm 3$ mmHg
- pulse	$\pm 5$ % of the reading
Power source	2 x AAA batteries 1,5V
Accessories	Wrist Watch Blood Pressure Monitor Model M2-W, 2 batteries (AAA), Storage Case, Instruction Book, Warranty Card, kit box.

\*technical alterations are possible!

## 8. Warranty

Your Automatic Blood Pressure Monitor Model M2-W is warranted for 3 years from date of purchase. This warranty includes the monitor and the cuff.

The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The warrantee is only valid upon presentation of the warrantee card which was correctly filled in and sealed.

## 9. Important information on blood pressure and its measurement

### 9.1. What is blood pressure?

Blood pressure is the pressure that your blood exerts on the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a “pump”, sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.


There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteries; and a diastolic (lower) one, which means the blood pressure between two heartbeats.

SYS blood pressure  
DIA blood pressure

Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors – time of day, person’s condition, physical or mental activity, environment, etc.

## Day-Night Fluctuations of the Blood Pressure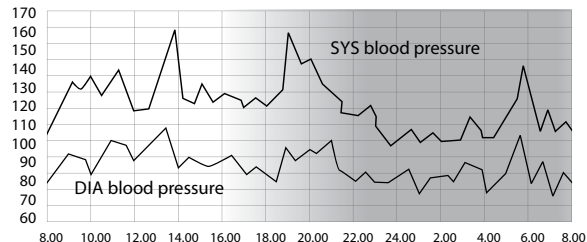

An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thick and less elastic. One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That’s why the self-control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitals (encephalon, heart, kidneys, blood vessels) breaks down. Without special treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

## 9.2. Which values are normal?

The world standard as for the norms of the blood pressure is the Classification\* of the World Health Organization (WHO):

Blood Pressure Category	Systolic Blood Pressure (mmHg)	Systolic Blood Pressure (mmHg)	Measures
Hypotension	< 100	< 60	Doctor's control
Optimum pressure	100 ... 120	60 ... 80	Self-control
Normal pressure	120 ... 130	80 ... 85	Self-control
High normal pressure	130 ... 140	85 ... 90	Self-control, mode of life correction
Mild hypertension	140 ... 160	90 ... 100	Consultation with doctor
Moderate hypertension	160 ... 180	100 ... 110	Consultation with doctor
Severe hypertension	> 180	> 110	Consultation with doctor

\* printed with curtailments

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of


hypertension development with advancing age.

### ATTENTION

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

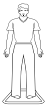When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.

### 9.3. What can be done to lower/maintain the level of blood pressure?

Recommendation		How much the systolic blood pressure can be lowered
- Try to maintain the weight which is considered standard for your age. Decrease overweight if any!		5 – 20 mmHg with every 10 kilos of weight reduction
- Avoid salt overuse (it's recommended to use no more than 6 g per day)!		2 – 8 mmHg
- Avoid eating fat food! Introduce to your diet some food that is rich in dietary fiber (fruits, vegetables)		8 – 14 mmHg

<ul style="list-style-type: none"> <li>- Refuse from smoking outright!</li> <li>- Use only limited amount of alcohol (bier – 350 g, or vine – 150 g, or vodka – 50 g)!</li> <li>- Restrain the usage of caffeine (coffee)!</li> </ul>		<p>2 – 4 mmHg</p>
<ul style="list-style-type: none"> <li>- After medical examination please regularly go in for sports, with the following restrictions: <ul style="list-style-type: none"> <li>• go in for endurance exercises not strengthening exercises.</li> <li>• do not train till complete exhaustion.</li> <li>• if you have any chronic diseases and/or you are older than 40 please consult your doctor before you start going in for sports.</li> </ul> </li> <li>- Spend at least half an hour out in the open air every day.</li> </ul>		<p>4 – 9 mmHg</p>

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