**(EN) ULTRASONIC NEBULIZER SYSTEM**

**MODEL COMPACT**  
USER'S MANUAL

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[www.dr-frei.com](http://www.dr-frei.com)

EN

**ULTRASONIC NEBULIZER**  
**AEROSOL THERAPY SYSTEM**  
model **COMPACT**  
INSTRUCTION MANUAL

Thank you for choosing **Dr. Frei**® Ultrasonic Nebulizer. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of **Dr. Frei**® Trade Mark.

Before starting to use the Ultrasonic Nebulizer please study the instruction book carefully. Following the rules stated in the instruction book will provide for the faultless operation of the Ultrasonic Nebulizer for a long period of time.

**CONTENT:**

1. Introduction
2. Recommendations
3. Spare parts description
4. Use instructions
5. Operation with car lighter adapter\*
6. Operations with rechargeable battery pack\*
7. Clean and maintenance
8. Air filter change
9. Accessories provided
10. Optional accessories
11. Trouble shooting chart
12. Technical specifications
13. Symbols key
14. Certifications
15. Warranty
16. What is the nebulizing therapy?

## 1. INTRODUCTION

Thank you for choosing our Ultrasonic Nebulizer designed to carry out aerosol therapy of acute and chronic diseases of upper and lower respiratory passages. This device allows for easy and effective transportation of big medicine doses directly to the disease centre.

We warrant you that this device has been produced using high quality materials and it complies with the safety rules in force both domestically and internationally.

The Ultrasonic Nebulizer operates on ultrasound which produces the drug solution vibration and turns it into mist, these tiny particles get deposited in the lungs and consequently, effective and quick results are achieved and no side-effects are caused.

This model comes with the exclusive Closed Water Chamber, easy to use: all you need is to pour the solution into the medication cup and it is ready to use.

It operates on three kinds of power supplies:

- mains supply
- car lighter adapter\*
- rechargeable battery pack\*

\* optional

Before starting to use the nebulizer system please study the instruction manual carefully. The instruction manual offers all information you need to correctly carry out aerosol therapy.

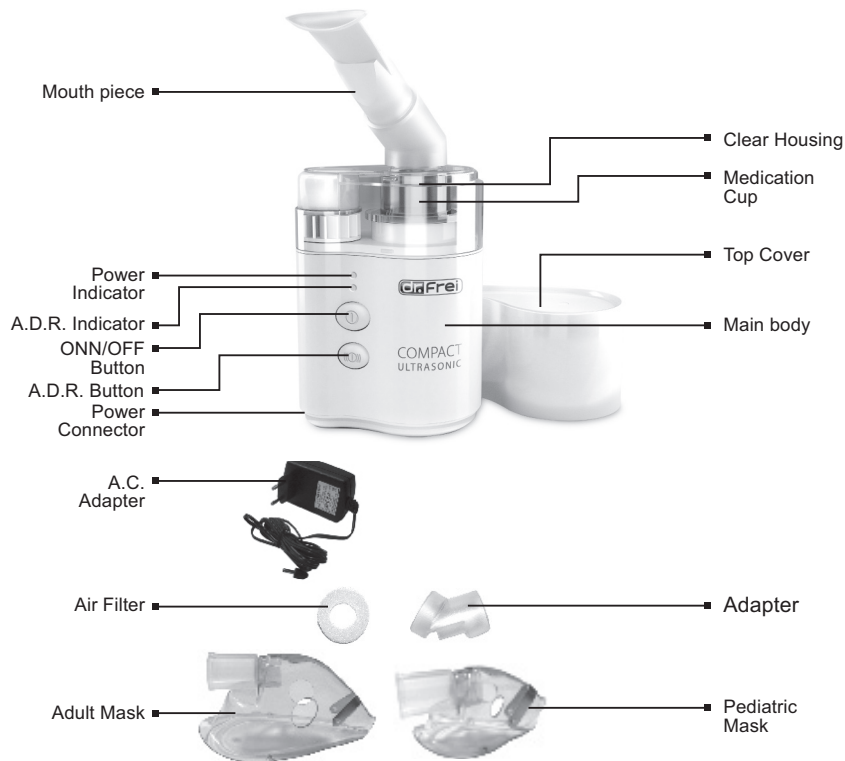
### **ATTENTION!**

Please, use only those medicines that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods prescribed without consulting your doctor.

## 2. RECOMMENDATIONS:

- Read the manual carefully before using the device and keep it in case you need further information in the future.
- Do not turn the device on if there is no drug solution into the medication cup.
- Disconnect the nebulizer when it is not being used.
- Do not tilt the device nor remove the medication cup cover while the device is operating. Use it on a flat surface.
- Do not remove the medication cup cover while the device is operating.
- Do not expose the nebulizer to a heat source.
- Do not pour more than 8 ml. of solution into the medication cup.
- Do not leave drug solution into the medication cup when the device is not being used.
- Do not scrape or pierce the medication cup cover.
- Do not touch the medication cup or the drug solution while the device is being used.
- Do not dip the nebulizer in water.
- Children may only use the device under adult supervision.
- To avoid infections and contagion the accessories should not be shared by different patients.  
They are intended for personal use.
- Use the unit intermittently. Maximum operating period: 30 minutes.  
Rest period: 30 minutes.
- Do not turn the device on without the cup cover.
- Do not use the device while driving.
- This device can only be repaired by the local authorized service agents.
- Only use nebulize solutions prescribed by a doctor and follow his instructions carefully.  
Do not use balm substances

### 3. SPARE PARTS DESCRIPTION

#### 4. USE INSTRUCTION

1. Take the unit and the AC adapter out of the box; carefully check the membrane placed at the bottom of the medication cup. If it is pierced or broken do not use the nebulizer and contact the authorized Service Centre.
2. Remove the clear housing and pour the medication into the medication container (8 ml. max.)
3. Place the basin cover lid and attach the accessory to be used (adult mask, child mask, and mouth piece).
4. Connect the power source to a proper mains socket.

##### *USE: Normal Mode*

After connecting the power PLUG to the equipment, a red light will switch on indicating that the unit is in standby. to start nebulizing, press the ON/OFF button (large button), the red light will turn green and the unit will start nebulizing. After approximately 4 minutes, the safety timer will be activated, the green light will turn red and the unit will stop. If you wish to continue with the nebulization, check that there is enough solution in the basin and then press the On/OFF button twice. This operation must be performed until completing the nebulizer use.

##### *A.D.R. Mode*

This intermittent operation mode enables the patient to synchronize the breathing pattern with the nebulizing frequency. taking 100% advantage of the nebulization. The patient must inhale during the emission and exhale when the unit stops, repeating the same pattern. To start or stop the A.D.R. system, simply press the lower button.

5. After use, disconnect the unit and the power source. Remove the basin cover lid and dry the solution remainder an absorbent cloth.

## 5. OPERATION WITH CAR LIGHTER ADAPTER (optional accessory)

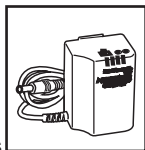
Connect the adapter to the car lighter and the other wire end to the unit. Then operate the nebulizer as per the instructions previously described.

OPTIONAL:  
Car lighter adapter

## 6. OPERATION WITH RECHARGEABLE BATTERY PACK (OPTIONAL)

1. When using the battery pack for the first time charge the batteries for 14 hours. Subsequent charging periods may vary from 6 to 12 hours.
2. Make sure the ON/OFF switch is in OFF position. Plug the power supply into the mains.
3. Connect the plug of the power supply to the socket placed at the bottom of the battery pack and charge the batteries without exceeding 24 hours.

OPTIONAL:  
Battery Pack

## 7. CLEAN AND MAINTENANCE

**IMPORTANT:** Clean the accessories before using the nebulizer for the first time or after each use following these instructions.

1. Wash all the pieces, except for the air filter, with warm water and detergent. Rinse with warm tap water to remove the detergent.
2. Dip the pieces into a clean bowl containing 3 parts of warm water and 1 part of vinegar during 30 minutes, disinfectant can also be used.
3. Wash your hands and take out the pieces from the disinfectant solution, rinse them with warm tap water and leave them on an absorbent piece of paper.
4. Keep the pieces into a hermetic bag. Do not dry them with a towel.
5. Clean the outer surface with a dry piece of cloth.

**WARNING:** In order to avoid infections, it is highly recommended to clean the pieces after each use. Only nebulize solutions prescribed by a doctor and follow his instructions carefully. Do not use balm substances.

## **8. AIR FILTER CHANGE**

1. The air filter has to be replaced if it turns dirty or torn.
2. Remove the air chamber from the medication cup cover.
3. take out the filter and place a new one.
4. Put all the pieces back.

## **9. ACCESORIES PROVIDED**

1. Pediatric mask -1
2. Adult mask - 1
3. Adapter -1
4. Air filters - 2
5. Mouth piece - 1
6. AC adapter - 1
7. Instruction manual and Warranty

## **10. OPTIONAL ACCESSORIES**

1. Car-lighter Adapter
2. Rechargeable Battery Pack

## 11. TROUBLE SHOOTING CHART

If something wrong happens while the device is working, please check and correct the following points:

Problem	Possible cause	Suggested solutions
Although the device is on, it does not work	<ol style="list-style-type: none"><li>1. Incorrectly plugged.</li><li>2. No power.</li><li>3. The AC Adapter is not connected.</li></ol>	<ol style="list-style-type: none"><li>1. Check the plug is properly connected.</li><li>2. Check the power is not gone out.</li><li>3. Check the AC Adapter is properly connected.</li></ol>
The mist does not come out	There is no solution into the cup.	Pour some drug solution into the medication cup.
Little mist is delivered	<ol style="list-style-type: none"><li>1. The air filter is dirty.</li><li>2. There is not enough solution into the cup.</li><li>3. Little power.</li><li>4. Air excess.</li></ol>	<ol style="list-style-type: none"><li>1. Change the filter.</li><li>2. Pour some drug solution into the medication cup.</li><li>3. Increase the power.</li><li>4. Regulate the air flow.</li></ol>
The device operates for a period and then turns off	Timer off.	Press the key twice and turn it on again.

Should the device not start working properly again, please contact the Service Centre; the address is stated in the Guarantee Card.

## 12. TECHNICAL SPECIFICATIONS

Method of nebulization	Ultrasonic vibration
Operating Voltage	100-240 V, 50-60 Hz, 12V, 1,25 A
Class	□
Current	15 W
Oscillation Frequency	2,5 MHz
Nebulization Rate	0,5 ml / min (minimum)
Particle Dimensions	1.5 to 5.7 (MMAD 3.8 $\mu$ m)
Medication cup maximum capacity	8 ml
Weight	260 g
Device dimensions	83,5 x 169 x 60 mm
Service Cycle	30 minutes on, 30 minutes off
Noise level (at 1 meter)	25 dB
Operating temperature and humidity	5 to 40°C , 30 to 95 %
Storage temperature and humidity	1 to 40°C, 10 to 95%

\* The Closed Water Chamber contains a solution of Benzalkonium Chloride at 0,01%.

\* The specification and the outer design are subject to change for improvement reason without previous notice.

### 13. SYMBOL KEY

((Ⓢ)) (( )) ON/OFF A.D.R.

Tape B

Read the instructions

AC

Class II

### 14. CERTIFICATIONS

Nebulizer Compact meets the highest international quality standards and was awarded the following Certificates:

- ISO 9001 Quality Systems (International)
- 13485 MEDICAL EQUIPMENT (European Community)
- 93/42/CE Directive (European Community)

### 15. WARRANTY

The ultrasonic nebulizer system device is covered by a 2-year warranty from date of purchase for any defects in the design or in the materials used. The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The manufacturer shall not be liable for any damages to people, property or pets, either directly or indirectly, caused by the non-compliance with the instruction book. The warrantee is only valid upon presentation of the warrantee card which was correctly filled in and sealed.

## **16. WHAT IS NEBULISING THERAPY?**

### *What is a nebulizer?*

The name of the device comes from the Latin word nebula that means fog, cloud. The nebulizer system is a device that turns a liquid drug into a fine aerosol. Aerosol is a drug suspension. So, nebulizing therapy is an inhalation therapy with the help of aerosols. Inhalation therapy has been used for treatment of lung diseases for a long time. The ancient civilizations of Egypt, India, China, the Middle East, Greece are known to have used inhalations of evaporations from menthol, eucalyptus, different plants.

For patients with lung diseases nebulizing therapy seems to be the most rational one since big drug doses come very fast to the respiratory tract directly.

### *How does the nebulizer system work?*

Nebulizer directs an aerosol containing larger and smaller particles to the respiratory tract. The gravity makes the larger particles accumulate on the walls of the upper respiratory passages while the smaller particles have enough time to reach the lower bronchial branches. Therefore the drug comes to all parts of the patient's bronchial tree, including the smallest bronchi and alveole, and gets into the blood vessels on the walls of the alveole in big therapeutic doses with tiny or none side effects. Some drugs are very slow to be absorbed from the alimentary canal or are affected dramatically by the first passage through the liver. In such cases the way out is to use inhalation therapy. Nebulizing therapy can be used by patients of any age, it's suitable for small children and infirms.

### *What is nebulizing therapy used for?*

Nebulizer systems are used by the patients with:

- acute respiratory diseases

- obstructive pulmonary disease
  - chronic inflammatory processes in respiratory passages such as chronic bronchitis, bronchial asthma (incl. chronic or unstable ones, exacerbation of asthma), chronic pharyngitis
  - respiratory passages infections
  - abnormalities in secretion quality and secreting in bronchi
  - different types of coughing
- Nebulizing therapy helps to decrease the level of inflammation activity, to improve respiratory passage drain function, to lessen the bronchismus, to make the edema smaller and to protect the mucous membrane, etc.

What are the advantages of nebulizing therapy compared to other means of drug delivery? Nebulizing therapy gives a possibility to quickly effect the inflammatory zone by big drug doses which increases the influence of the treatment and reduces the risk of side effects to other organs. Nebulizing therapy is the easiest way of inhalation treatment and it is also an economical means of treatments from the point of view of purposeful use of drug. Aerosol treatment with the help of nebulizing systems can be used for children, elderly and infirm patients.

Please, use only those drugs that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods/frequency prescribed without consulting your doctor.

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■ SAN UP S.A. Ruta 8 No 2967, 1650 San Martin, Buenos Aires, Argentina
Manufactured by the order of: Medhouse Swiss GmbH, Tramstrasse 16, 9442 Berneck, **Switzerland**